

iCassette™

One Step Drug Test

Detects 1-12 drugs

Laboratory Accurate

Runs on 3mL of Specimen

Easy to use



Fast
Results

FDA
Cleared

Cost
Effective

Results in Minutes...



Nine Panel Shown Above

MULTI

Catalog N
Lot: DOA3

For in vit
Store at any temp

Instan

iCassette
Drug Screen

MULTI DRUG TEST DEVICE

(AMP/COC/THC)+(AMP/OPI/PCP)+(BZO/BAR)+MTD
Catalog Number: i-DOA-1195
Lot: DOA304023 Exp: 2005/03

For In vitro diagnostic use only.
Store at any temperature from 2 - 30°C (35 - 86°F)

MULTI DRUG TEST DEVICE

For In vitro diagnostic use only.
Store at any temperature from 2 - 30°C (35 - 86°F)

iCassette™

One Step Drug Test


Abbreviation	Drug Name	Cut-Off (Cut Off Levels in ng/mL)
AMP	Amphetamine	1000
BAR	Barbiturates	300
BZO	Benzodiazepines	300
COC	Cocaine	300
THC	Marijuana	50
MTD	Methadone	300
mAMP	Methamphetamine	1000
MDMA	Methylenedioxymethamphetamine	500
MOR	Morphine	300
OPI	Opiates	2000
OXY	Oxycodone	100
PCP	Phencyclidine	25
PPX	Propoxyphene	300
TCA	Tricyclic Anti-Depressants	1000




Five panel shown above

PROCEDURE CARD FOR SINGLE AND MULTI DRUG TEST DEVICES


- 1** Remove the test device from sealed pouch.

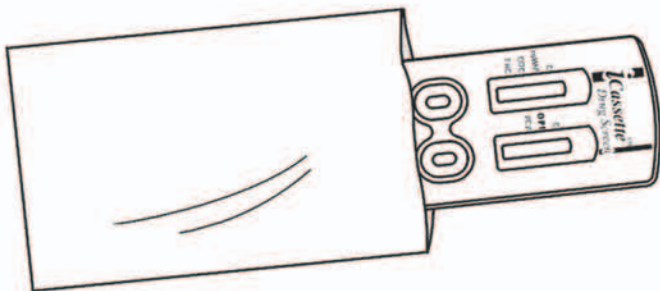

- 2** Hold the dropper vertically and transfer 3 full drops of urine to the specimen well of the test device.

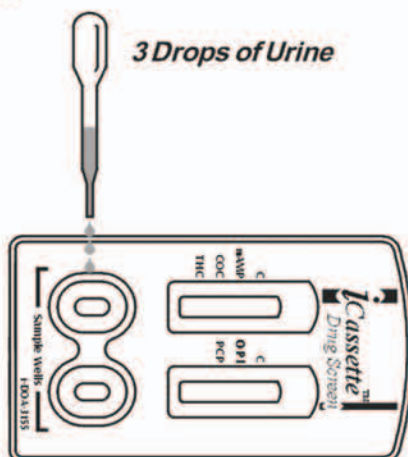

- 3** Read results at 5 minutes.

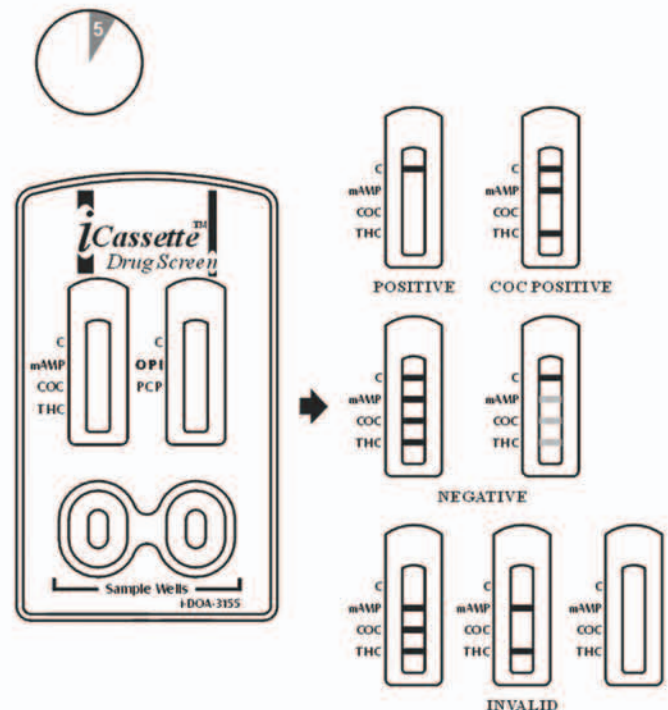

Do not interpret results after 10 minutes.