


iScreen™


One Step Drug Card

Abbreviation	Drug Name	Cut-Off (Cut Off Levels in ng/mL)
AMP	Amphetamine	1000
BAR	Barbiturates	300
BZO	Benzodiazepines	300
COC	Cocaine	300
THC	Marijuana	50
MTD	Methadone	300
mAMP	Methamphetamine	1000
MDMA	Methylenedioxyamphetamine	500
MOR	Morphine	300
OPI	Opiates	2000
OXY	Oxycodone	100
PCP	Phencyclidine	25
PPX	Propoxyphene	300
TCA	Tricyclic Anti-Depressants	1000

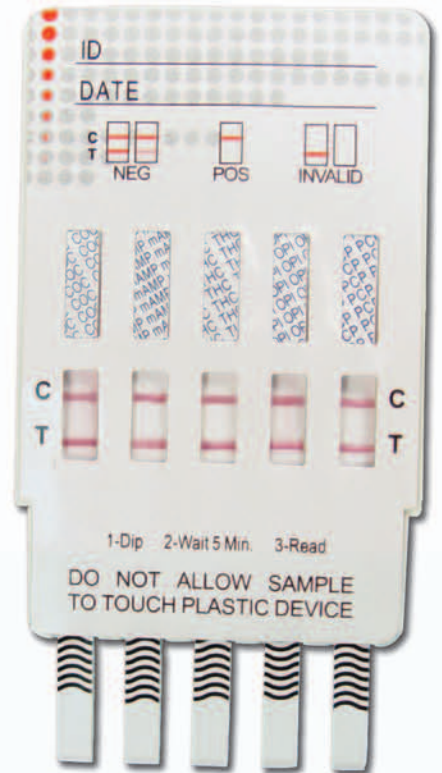

Five panel shown above

PROCEDURE CARD FOR SINGLE AND MULTI DRUG SCREEN TEST CARDS

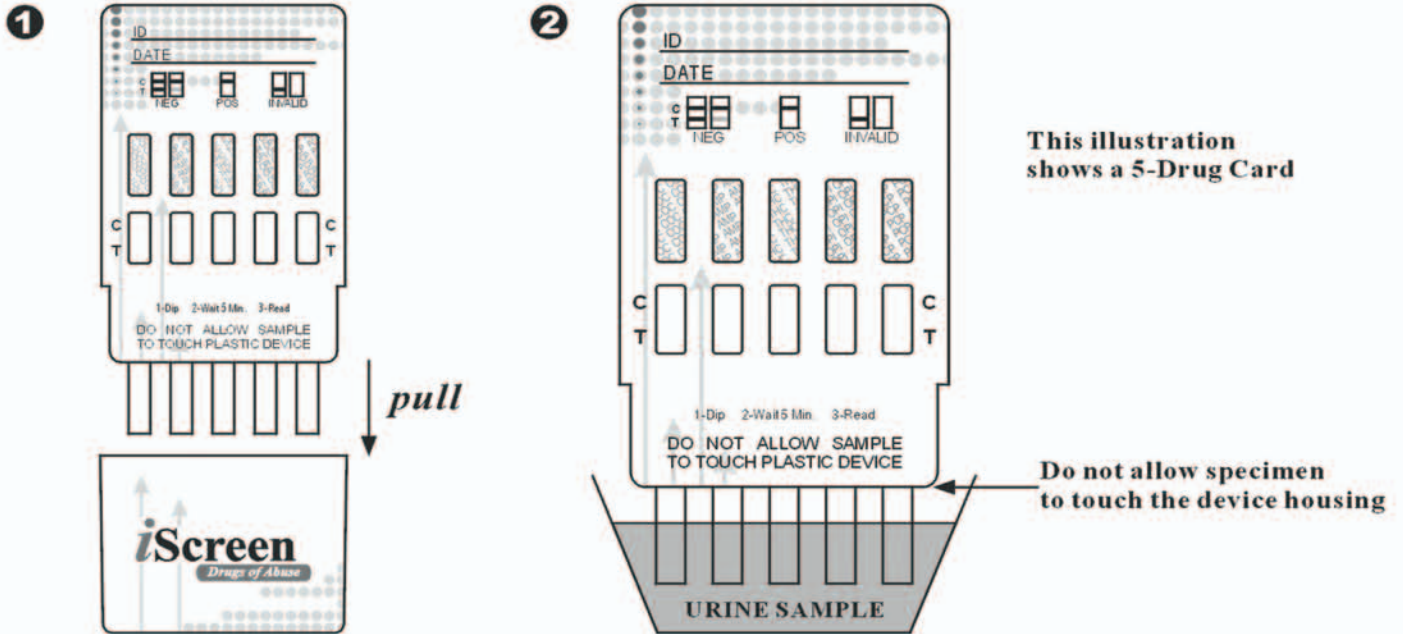

1. Remove iScreen test from foil pouch at room temperature. Take off the cap revealing the testing strips.
2. Immerse test vertically in the urine sample for at least 10-15 seconds. Do not allow the specimen to touch the device housing.
3. Read at 5 Minutes.

4. Interpretation of Results.



Do not interpret results after 10 minutes.